



SEQ944 Magnifying Lamp



Thank you for purchasing the SEQ944 Magnifying Lamp.

Function:

This Magnifying Lamp features a 3 diopter lens and can be used for clinical examinations of the skin. With a clamp holder, this magnifying lamp can be mounted to a table top for convenient operation.

Operating Instructions:

1. Using the clamp, attach to a table top, making sure it is securely fixed.
2. Plug in power.
3. Turn on power switch.
4. Adjust the spring arm to the desired position.

Specification:

Voltage : **220V/50Hz**

Power: **22 watts**

Safe operation and cautions:

1. The spring adjuster is designed intended for the spring flexibility. When the arm is lifted, the friction makes a certain noise, this is completely normal.
2. Do not extend the arm straight and leave it at this position for a long period of time. This can cause the spring to loose flexibility. When not in use, adjust it back to the default position.



LVD Certificate No. :ASZ091104030S. EMC Certificate :ASZ091109050E

**Salonserve Hair & Beauty Supplies, Lexicon House, Station Road,
Irthlingborough, Northamptonshire, NN9 5QF - 01933 410 022**

www.salonserve.com