

# SalonSERVE®

## User Manual




- Offer desincrustation and iontophoresis facial treatments.
- Has a digital timer, digital power control and digital reverse polarity.
- Slimline design can be moved around with ease, great for small rooms.
- Comes complete with:  
Client electrode; Ball electrode; 2 x Roller electrode.
- Size: 24cm x 20cm.
- 6 months warranty.

## Galvanic

SEQ642

# The Galvanic machine

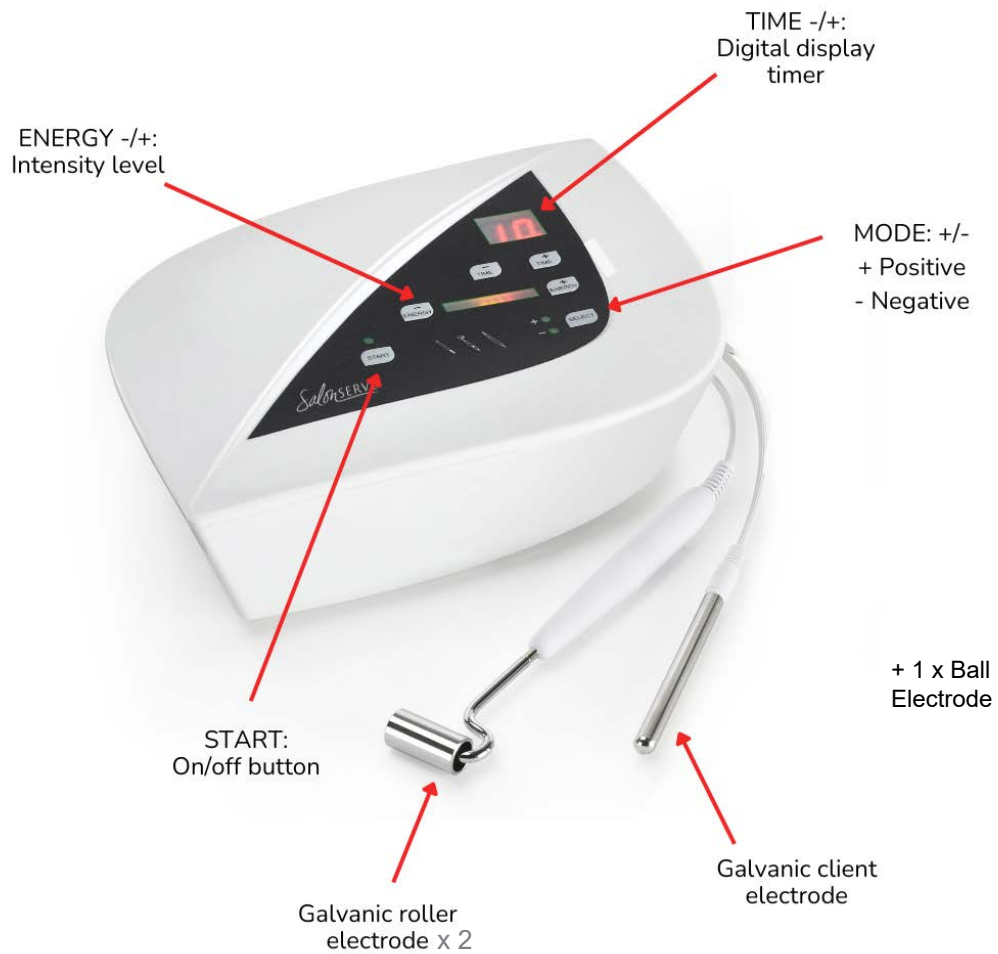

## Contents

Introduction .....	3
Benefits of the Galvanic machine .....	3
Contra Indications .....	3
The Galvanic electrodes and their uses .....	4
The Galvanic gels .....	4
Operating instructions .....	5
Galvanic facial treatment procedure .....	6
Technical specifications and machine care .....	7

# Introduction

The Salonserve galvanic machine enables you to offer a variety of treatments to the skin. These include deep cleansing, soothing, decongesting, hydrating and warming treatments for both the face and body.



## Galvanic

The Salonserve galvanic facial machine is a low frequency electrical treatment unit which uses direct current to produce deep down chemical effects into the dermis and onto the surface of the skin.

The Salonserve galvanic facial machine enables you to offer a variety of deep cleansing and rehydrating treatments to your clients. Using a combination of galvanic and desincrustation low voltage current the galvanic facial machine provides gentle stimulus for all skin types. Treatments can be tailor made and adapted to suit your client's needs. In galvanic mode the facial unit provides a deep cleansing treatment for oily and combination skins. It also works on deep cleansing seborrhoeic and acne skins and stimulates grey, sluggish skin.

When used as an iontophoresis treatment the facial unit calms and soothes the skin. The positive polarity also restores the skin's acid mantle and tightens and firms the skin leaving it looking calm and radiant at the end of the treatment.

The Salonserve facial galvanic machine can be used as a stand-alone treatment or combined with other electrical treatments such as steaming, high frequency and brush cleansing. By combining treatments in this way each client can be given a tailor made course of treatments to treat their particular skin complaint.

## Benefits of the Galvanic Facial Machine

### Desincrustation method

- Deep cleanses.
- Stimulates sluggish skin
  - Saponifies sebum
  - Breaks down keratin
  - Increases vasodilatation thus stimulating lymph removal, cell metabolism and oxygenation.

### Iontophoresis method

- Hydrates and moisturises skin
  - Propels products into the skin
  - Restores skin's acid mantle
  - Tightens and firms the skin
  - Soothes nerve endings
  - Revitalises tissues
  - Refines enlarged pores and improves texture of the skin.

## Contra Indications

- Skin diseases or disorders
- Recent scar tissue
- Pregnancy
- Excessive metal fillings
- Sinus blockages
- Tense nervous clients
- Hairy areas
- Hypersensitive skin
- Highly vascular skin
- Epilepsy
- Diabetes

## The Galvanic electrodes and their uses

The roller electrodes are used for the main part of the treatment and are used to glide smoothly across the cheeks, forehead, sides, jaw line, neck and décolleté areas. The rollers are used for both desincrustation and iontophoresis alike.

The ball electrode is for use around the nose and chin. Hold the electrode like a pen and use small circular movements to work closely around the crevices of the nose and chin.

Wrap damp cotton wool around the inactive electrode before giving to the client to hold to aid conduction.



## Galvanic Gels


Salonserve offer a desincrustation gel to be used for all skin types and a collagen iontophoresis gel to choose from depending on skin type. The gels are an ideal blend for each skin type using natural plant extracts. The gels do not need to be mixed together; only one gel is needed to perform the treatment.

- **Desincrustation galvanic gel - 500ml - code: GGD**  
A deep cleansing treatment gel designed to clear blocked pores and boost circulation.
- **Collagen galvanic gel - 500ml - code: GGC**  
Blended with collagen and elastin, this gel has high moisturising properties and is ideal for dry, mature skin types. For use in iontophoresis.

# Galvanic treatment operating instructions

1. Plug in mains lead and check the intensity control (ENERGY) is at minimum.
  2. Plug in rollers into socket and plug in inactive electrode into wire with small metal end. Wipe over electrodes with surgical spirit.
  3. Check that intensity control is at min and switch on.
  4. Apply desincrustation gel to the skin and place a wad of damp cotton wool around the inactive electrode before giving it to the client to hold.
  5. Apply petroleum jelly around the eyes and corners of the mouth to prevent flashing.
  6. Ensure the client has removed all jewellery before starting treatment.
  7. Check the machine is set to MODE: negative.
  8. Holding the roller in one hand place it on the client's skin and begin moving roller back and forth. At the same time slowly increase the intensity. When the client begins to feel a tingling sensation turn the intensity down slightly and begin treatment. Work slowly over the face with the roller, then change to the roller ball electrode to work around the nose and chin. Desincrustation time should not exceed 10 minutes, work for just 2-3 minutes on a sensitive or dry skin. Turn the intensity down and switch off before the next part of the treatment.
  9. Remove the desincrustation gel thoroughly with damp cotton wool.
  10. Tone skin.
  11. Apply appropriate iontophoresis gel to the skin.
  12. Apply petroleum jelly around the eyes and corners of the mouth to prevent flashing.
  13. Change the machine to MODE: positive polarity.
  14. Holding the roller in one hand place it on the client's skin and begin moving roller back and forth. At the same time slowly increase the intensity. When the client begins to feel a tingling sensation turn the intensity down slightly and begin treatment. Work slowly over the face with the roller, then change to the roller ball electrode to work round the nose and chin. Iontophoresis time should not exceed 12 minutes, work for just 2-3 minutes on an oily or acne skin just to replace the acid mantle.
- ALWAYS REMOVE THE ROLLERS BEFORE THE MACHINE POWERS OFF; DO NOT POWER OFF ON THE CLIENT.**
15. Remove the iontophoresis gel thoroughly with damp cotton wool before commencing rest of treatment. If performing a combined galvanic, high frequency treatment move onto high frequency instructions.

## Galvanic Roller Movements

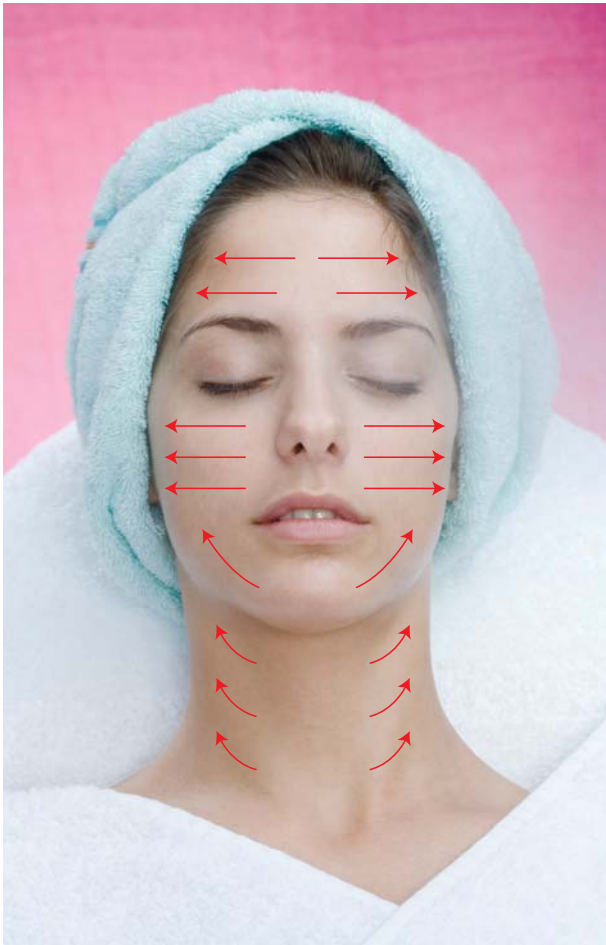

**ALWAYS REMOVE THE ROLLER BEFORE THE MACHINE POWERS OFF; DO NOT POWER OFF ON THE CLIENT.**

## Galvanic Facial Treatment Procedure

**Time: 45 mins - 1 hour**

**RRP: £45- £65**

This is an excellent deep cleansing facial, which is especially suitable for oily, acne or combination skin types. The galvanic treatment can be used on its own or as part of an electrical facial session as shown below. When performing desincrustation treatment time varies according to skin type. Oily skins: 10-12minutes. Dry/dehydrated: 3-5 minutes. When the polarity is switched over to positive work for 2-3 mins on oily skins and up to 12 minutes on a dry or mature skin.

1. Perform double cleanse then do skin assessment on client and discuss appropriate treatment plan with client.

2. Steam skin and/or brush cleanses to remove any surface congestion.
3. Apply desincrustation gel to the skin and place a wad of damp cotton wool around the inactive electrode before giving it to the client to hold.
4. Apply petroleum jelly around the eyes and corners of the mouth to prevent flashing.
5. Ensure the client has removed all jewellery before starting treatment.
6. Check the machine is set to negative.
7. Using the rollers to work over the skin for the appropriate amount of time.
8. Remove the desincrustation gel thoroughly with damp cotton wool.
9. Tone skin.
10. Apply appropriate iontophoresis gel to the skin.
11. Apply petroleum jelly around the eyes and corners of the mouth to prevent flashing.
12. Change the machine to positive polarity.
13. Using the rollers to work over the skin for the appropriate amount of time. **ALWAYS REMOVE THE ROLLER BEFORE THE MACHINE POWERS OFF; DO NOT POWER OFF ON THE CLIENT.**
14. Remove the iontophoresis gel thoroughly with damp cotton wool.
15. A high frequency treatment works well at this stage of the facial otherwise move on to facial massage.
16. Now apply an appropriate mask and leave for 10-15 minutes.
17. Remove mask with warm water and sponges, tone skin and apply eye and face creams to finish.

# Technical & Machine Care

## Technical Specifications

Power supply: 220 - 240 volt  
Hertz: 50 - 60Hz

G.W.: 2.2kg  
Size: 24cm x 20cm

## Machine Care

- Do not immerse any parts of the machine or galvanic electrodes in water.
- Clean the handset with surgical spirit on cotton wool between each client.
- Clean the unit with a damp soft cloth.

## Notes

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

Salonserve Hair & Beauty Supplies LTD, Lexicon  
House, Irthlingborough, Northants, NN9 5QF  
Tel: 01933 410022

[www.salonserve.com](http://www.salonserve.com)